BP-200 plus

Accuracy and performance in stress test blood pressure measurement

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SCHILLER
The Art of Diagnostics

BP-200 plus – Accuracy and performance in stress test blood pressure measurement

We focus on blood pressure, so you can focus on your patient.

The BP-200 plus blood pressure monitor is designed to automatically measure and display a patient's systolic and diastolic blood pressure and send the measurements to a stress test system. The BP-200plus also measures and displays heart rate and SpO₂ (option) to create a complete and seamless diagnostic test centre. The BP-200 plus is specifically designed for stress test environments using a treadmill or ergometer and is also ideal in research environments because raw data can be saved on an SD-Card.

The BP-200 plus excels where other BP monitors fail by using Schiller's proprietary SCHILLER K-Sound Analysis (SKA). SKA was developed as a result of Schiller's pioneering work in 24-Hour Ambulatory Blood Pressure Monitoring. It was this experience that provided Schiller's research and development team with invaluable insights into the relationship between many of the variables involved in determining the blood pressure of a patient in motion. These variables include, but are not limited to, the patient's ECG, K-sounds, overall body

shape, foot strike characteristics, arm movements, along with treadmill and ergometer noise. With SCHILLER K-Sound Analysis, Korotkoff sounds are distinguished from artifact noise so that blood pressures are measured accurately with greatly reduced false measurements.

With the BP-200 plus "hands free" interface, blood pressure readings can be initiated automatically by your stress ECG system at user defined intervals. After taking a blood pressure measurement, the monitor automatically sends the results to your stress ECG system for immediate display and printout. The BP-200 plus gives you the freedom to concentrate on your patient's performance without the need to manually initiate blood pressure measurements.

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System:

Dimensions: 222 x 155 x 96 mm (l x w x h)

Weight: 1,08 kg (including SPO₂, ECG Amp, excluding batteries)

Power supply: External supply+9 VDC output @ 0-1,7 amps

LCD: Backlit graphical LCD with multi-language menu

Interface:

- ECG and RS-232 interface for SCHILLER ECG stress test systems

- USB interface

Programming: Menu guidance 5 buttons

Safety standards: IEC/EN 60601-1; UL 60601-1, C22.2 No. 601.1-M90; IEC/EN 60601-1-2 (EMC)

Protection Class: I according to IEC/EN 60601-1 (option: internal power supply)

Applied Part: BF/CF according IEC/EN 60601-1

Conformity: CE 0123 according Directive 93/42/ECC (Medical Devices)

Classification: IIa according Directive 93/42/EEC

Measurement:

Methods of measurement:

- **Primary:** SCHILLER K-Sound Analysis with or without QRS trigger signal for stress test

- **Secondary:** Oscillometric with smooth linear deflation for resting measurements

Measurement range:

- Systolic: 50 – 270 mmHg

- Diastolic: 20 – 150 mmHg

- Heart rate: 40 – 250 BPM

Measurement intervals: 2 - 20 min.

Memory capacity: 200 measurements

QRS trigger: Signal from external ECG source

Sampling intervals:

On command from ECG stress test system, manual or automatic measurement periods from 2 to 20 minute intervals

Indicator for the user: On screen message and audio tone for user defined ranges

Optional unit version:

Built in SPO₂ module and internal ECG amplifier (to generate the QRS trigger signal to support the blood pressure measurement)

Standard accessories:

- Cuff (M)
- User guide
- Velcro fixing pads to secure cuff
- Adhesive pads to fix the microphone
- External power supply
- Head phone
- Communication cable to stress test system
- QRS-trigger cable

Optional accessories:

- Several cuff sizes
- Batterie charger
- Rechargeable batteries
- SPO₂ with cable and ear-sensor
- Patient cable for internal ECG Amplifier (IEC coding, Code 1 / AHA coding, Code 2)
- Mini SD card for software update and additional storage of the raw data

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SCHILLER AG
SCHILLER ESPAÑA, S.A.
SCHILLER-Reomed AG
SCHILLER America Inc.